



Microbiology

MYCOPLASMA CONTROL



UREAPLASMA UREALYTICUM AND MYCOPLASMA HOMINIS QUALITY CONTROL STRAINS

- ▶ Meets your **quality management requirements** (compliant with ISO 15189 : 2007 recommendations).
- ▶ Relevant results **complying with MIC values** in liquid medium.
- ▶ Identification of strains performed on **A7 agar plates, U9 and Shepard media**.
- ▶ **Compatible** use with all **MYCOFAST®** products.
- ▶ **Long shelf life** of 12 months .

Easy procedure

- 1 Reconstitute the U.u or M.h. strain with 1mL of sterile distilled water.
- 2 Dilute the bacterial suspension into the transport medium (UMMt) or culture medium (UMM).
- 3 Inoculate the tray or microplate.

REF.	NAME	QTY
00900	Mycoplasma Control	6 vials of each strain (12 total)



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.



: ELITech Microbio (France)



Headquarters
Australia
BENELUX

+33 1 41 45 07 10 France
+1800 815 098 New Zealand
+31 313 430 500 United States

+33 4 83 36 10 82
+0800 555 611
+1 435 0752 6011



www.elitechgroup.com
em-marketing@elitechgroup.com