


# ELITex Bicolor RF WAALER-ROSE Bicolor

SEROLOGY RAPID TESTS



2 complementary slide agglutination tests for the rapid qualitative or semi-quantitative detection of IgM **RHEUMATOID FACTOR** in serum.

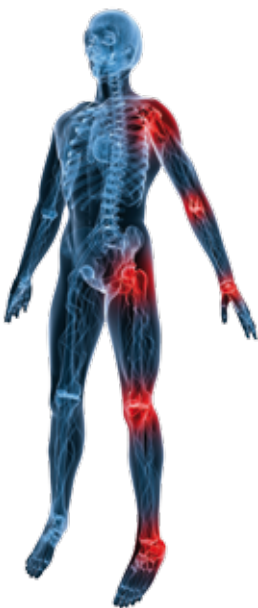

- ◆ **Easy reading: bi-colored reactions**
- ◆ Complete kits (controls included)
- ◆ Simple and fast protocol (2 minutes)
- ◆ Cost effective test
- ◆ Long shelf life

# SEROLOGY RAPID TESTS

## ELITex Bicolor RF - WAALER-ROSE Bicolor


### PRINCIPLE:

Latex particles (ELITex Bicolor RF) or Red Blood Cells (WAALER-ROSE Bicolor) are sensitized by gamma-globulins (Human IgG for ELITex Bicolor RF / Animal IgG fraction for WAALER-ROSE Bicolor) and added to the tested serum. If Rheumatoid Factor is present in the serum, it will agglutinate and be revealed by an easy-to-read **bi-colored** reaction.

### METHODOLOGY:

1. Add Serum or Control(s)  
= 10 µL for RF, 25 µL for WR
2. Add one drop of Test Reagent

### S I M P L E A N D F A S T

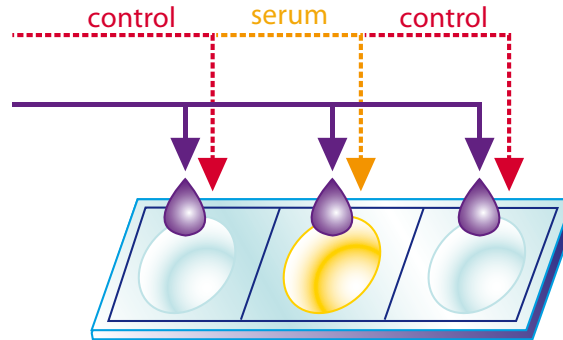

3. Slow rotary motion



#### POSITIVE REACTION:

##### ELITex Bicolor RF:

Red agglutinates on a blue background

##### WAALER-ROSE Bicolor:

Brown agglutinates on a blue background

#### INTERPRETATION:

Presence of Rheumatoid Factor

+

2  
min

-



#### NEGATIVE REACTION:

##### ELITex Bicolor RF:

Homogeneous purple suspension

##### WAALER-ROSE Bicolor:

Homogeneous green-brown suspension

#### INTERPRETATION:

Absence of Rheumatoid Factor

### PRODUCT RANGE:

REF.	NAME	QTY	SENSITIVITY	THRESHOLD (Shown on the box label)	SPECIFICITY
04165 / 04166	ELITex Bicolor RF	50 / 100tests	98.75%	≈ 8 IU / mL	98.37%
04175 / 04176	WAALER-ROSE Bicolor	50 / 100 tests	98.79%	≈ 8 IU / mL	98.79%

### AND REINFORCE YOUR DIAGNOSIS WITH:

04115	ELI.H.A RF	20 x 11 tests	97.6%	≈ 0.2 IU / mL	97.4%
-------	------------	---------------	-------	---------------	-------

- ◆ QUANTITATIVE DETERMINATION of Rheumatoid Factor in serum by Indirect Hemagglutination
- ◆ DETECTION AND ELIMINATION OF INTERFERENCES due to natural anti-sheep agglutinins

### Distributor Contact Information:

Please contact your sales representative for product availability in your country

Headquarters  
Belgium  
Brazil  
France  
Italy  
Middle East & Africa

T: +33 1 41 45 07 13  
T: +32 9 2820 531  
T: +55 27 3025 1415  
T: +33 4 94 88 55 00  
T: +39 02 48 40 35 42  
T: +971 4 375 2744

Serbia  
Switzerland  
The Netherlands  
UK  
United States

T: +381 11 2467119  
T: +41 26 663 86 60  
T: +31 313 430574  
T: +44 1442 869320  
T: +1 609 216 736  
855-ELITECH (855-354-8324)



ELITechGroup

SOLUTIONS

tailored to your needs

info@elitechgroup.com • www.elitechgroup.com