

Attention User: Before using this product, please read the following important information:

The Selectra Pro application for μ ALBUMIN IP reagent kit (IMAL-0400) has been improved. This modification requires:

- the uploading of a modified instrument application, which is available in the ELITechGroup Clinical Systems Portal.
- adjusted values for μ ALBUMIN IP CALIBRATOR SET and μ ALBUMIN IP CONTROL I & II as provided below.

μ ALBUMIN IP CALIBRATOR SET	Level	μ ALBUMIN IP Code: IMAL-0400	
		Values (mg/dL)	Values (mg/L)
20-4024	Cal 1	2.10	21.0
	Cal 2	5.30	53.0
	Cal 3	10.60	106.0
	Cal 4	21.20	212.0
	Cal 5	42.40	424.0
21-4025	Cal 1	2.20	22.0
	Cal 2	5.40	54.0
	Cal 3	10.80	108.0
	Cal 4	21.60	216.0
	Cal 5	43.10	431.0

μ ALBUMIN IP CONTROL I	μ ALBUMIN IP Code: IMAL-0400	
	Values (mg/dL)	Values (mg/L)
20-4023	2.95 (2.18 - 3.72)	29.5 (21.8 – 37.2)
21-4043	2.75 (2.04 - 3.47)	27.5 (20.4 - 34.7)

μ ALBUMIN IP CONTROL II	μ ALBUMIN IP Code: IMAL-0400	
	Values (mg/dL)	Values (mg/L)
20-4022	21.88 (16.19 - 27.57)	218.8 (161.9 – 275.7)
21-4044	20.99 (15.53 – 26.45)	209.9 (155.3 – 264.5)