




# Microbiology

## MYCOFAST® REVOLUTION ATB+

Urogenital mycoplasma diagnosis  
**DIRECT METHOD**

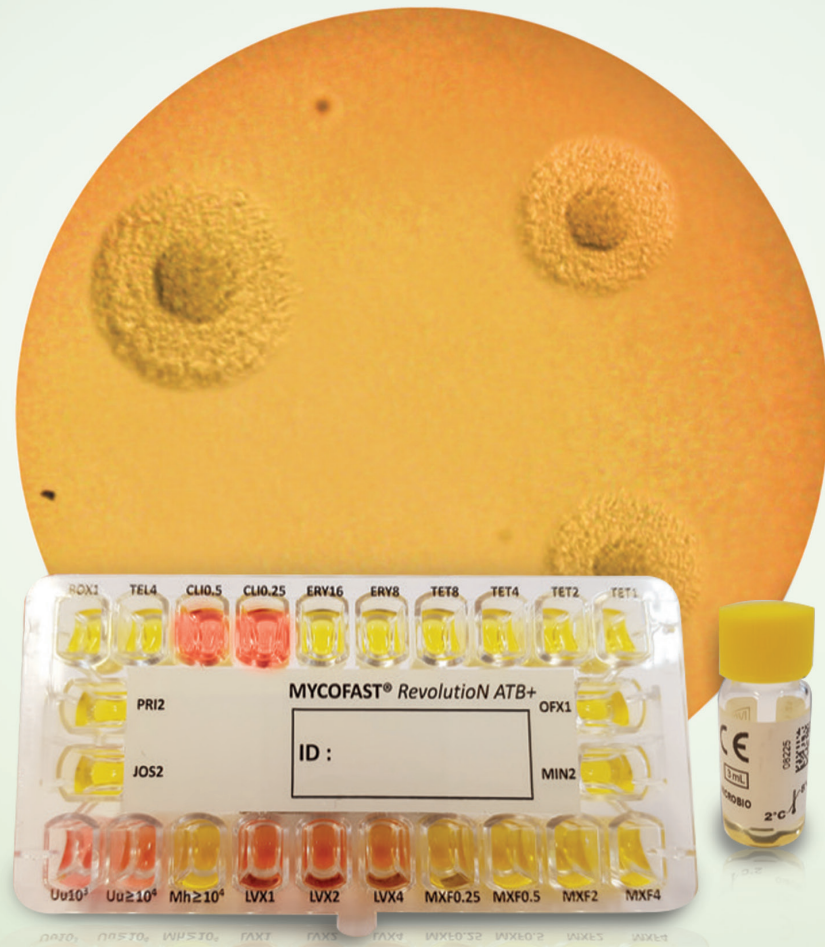

### EXPANDING THE SCOPE OF TREATMENT OPTIONS!

*Detection, Enumeration, Identification and Antimicrobial Susceptibility Testing*

- ◆ Extended panel of antibiotics tested, in full compliance with worldwide recommendations
- ◆ Complete diagnosis in an All-in-1 product
- ◆ The right pathologic threshold tested for each specimen type
- ◆ Fast results for clinicians
- ◆ Visual, colorimetric reading

# MYCOFAST® REVOLUTION ATB+



## ► BENEFITS

### ◆ Complete diagnosis in 24h\*

- ◆ Detection, Enumeration
- ◆ Identification
- ◆ Antimicrobial Susceptibility Testing

### ◆ Cutting-edge performance

#### Sensitivity

- ◆ Detection and enumeration of *U. urealyticum* at  $10^3$  and  $\geq 10^4$  CCU/mL
- ◆ Detection and enumeration of *M. hominis* at  $\geq 10^4$  CCU/mL
- ◆ The right pathologic threshold tested for each specimen type

#### Specificity

- ◆ Identification secured by checking *U.u./M.h* natural resistance to antibiotics (Clindamycin and Erythromycin)

#### Reliability

- ◆ 6 antibiotics (TEL/LVX/MXF/E/CM/TE) tested in compliance with CLSI guidelines (Clinical & Laboratory Standards Institute)
- ◆ 5 other antibiotics tested (MIN/OFX/ROX/PRI/JOS)

#### Fast results

- ◆ Integrated growth activator for *M. hominis*


### ◆ Ease of use

- ◆ Ready-to-use reagents
- ◆ Each well pre-filled with growth medium
- ◆ Visual colorimetric reading
- ◆ No instrument required

### ◆ Flexibility and Safety

- ◆ Optimized transport medium: Specimens preserved up to 72 hours before analysis
- ◆ Clip-on closing system for the tray

## ► METHODOLOGY

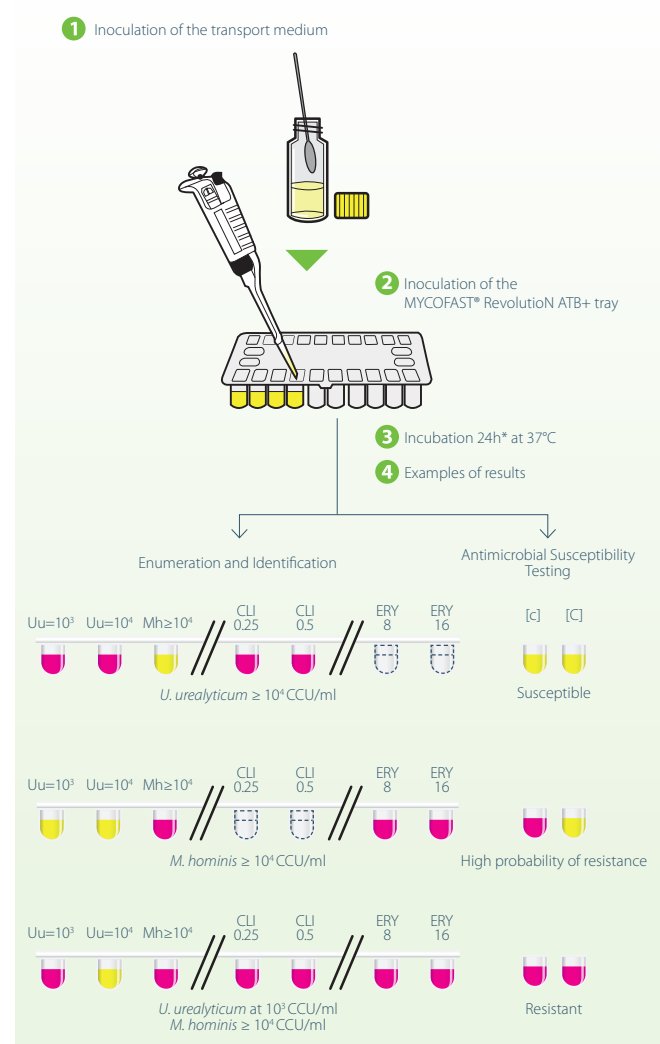

\* Reading of all samples in 24h.  
Only read liquid samples having negative results within 24h.

## ► PRODUCT RANGE

REF.	NAME	QTY	PURPOSE
00070	MYCOFAST® RevolutionN ATB+	25 tests	Complete kit
00061	UMMt RevolutionN (optional)	50 vials	Additional transport medium
00073	COMPLEMENT MYCOFAST® RevolutionN ATB+ (optional)	25 tests	Additional test trays
00064	MYCOPLASMA Stabilizer (optional)	100 tests	Additive for sample storage > 56 hours
00900	MYCOPLASMA Control (optional)	12 tests	U.u. and M.h. control strains



### WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.

#### Headquarters

Australia  
BELUX  
Brazil  
France  
Italy

#### +33 1 41 45 07 10

1800 815 098  
+32 92 820 531  
+55 27 3025 1415  
+33 4 83 36 10 82  
+39 011 67 61 913

New Zealand  
Serbia  
The Netherlands  
UK  
United States

0800 555 611  
+381 11 2467119  
+31 313 430 500  
+44 1442 869320  
+1 435 0752 6011



: ELITech Microbio (France)



**ELITechGroup**

EMPOWERING IVD

[www.elitechgroup.com](http://www.elitechgroup.com)  
[info@elitechgroup.com](mailto:info@elitechgroup.com)