




Microbiology

ELI.H.A RANGE

Serodiagnosis

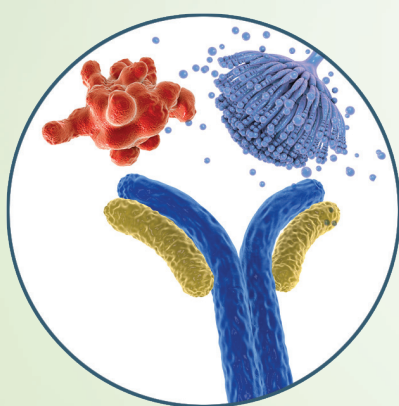

ELI.H.A RANGE: INDIRECT HAEMAGGLUTINATION (I.H.A)

Complete kits with liquid ready-to-use reagents

Results in 2 hours and reliable overnight

Mycology

- ◆ Candidiasis
- ◆ Aspergillosis

Parasitology

- ◆ Bilharziosis (Schistosomiasis)
- ◆ Amoebiasis
- ◆ Hydatidosis (Echinococcosis)
- ◆ Distomiasis
- ◆ Toxoplasmosis


ELI.H.A RANGE

► PRINCIPLE

◆ ELI.H.A RANGE IS BASED ON INDIRECT HAEMAGGLUTINATION TESTING

Sensitized sheep red blood cells (RBC) are coated with antigens and added to the test serum. If the specific antibodies are present in the serum, they will agglutinate with the red blood cells and a red/brown deposit will appear on the bottom of the well. With a non-reactive serum, no agglutination occurs with the sensitized cells and they drop to the bottom of the well, forming a compact button.


Example: Serum positive at a dilution of 1:640

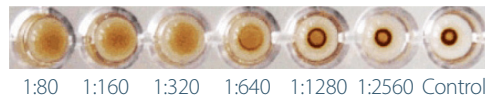

► METHODOLOGY

◆ A SINGLE PROTOCOL FOR ALL OUR KITS

1. First serum dilution
2. Microplate preparation
3. Serial dilution of serum & preparation of the "serum control" well
4. Addition of **sensitized RBC** and **non sensitized RBC**
5. Gently mix the plates & let stand for 2 hours before reading

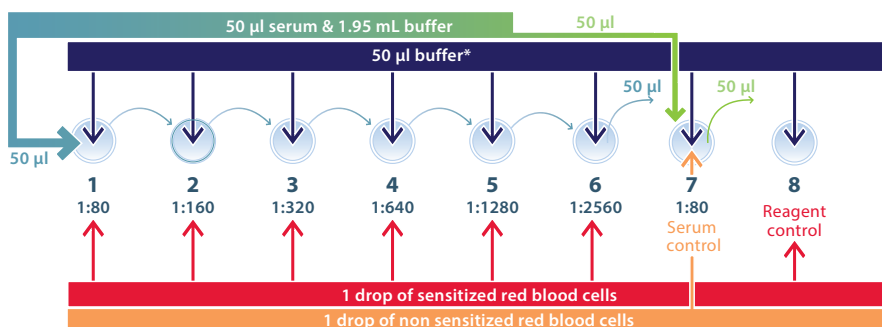

*Anti-Toxoplasma gondii IgM can be detected by adding 2-mercaptoethanol (not provided) to the buffer.

► KEY BENEFITS

- ◆ Long shelf life (> 12 months)
- ◆ Simple protocol
- ◆ Instrument-free solution
- ◆ Cost effective
- ◆ Results in international units for ELI.H.A Toxo

► PRODUCT RANGE

	REF.	NAME	DIAGNOSIS OF	QTY	SENSITIVITY	SPECIFICITY
Mycology	44600	ELI.H.A Candida	Candidiasis	120 Tests	85.4%	97.9%
	44602	ELI.H.A Aspergillus	Aspergillosis	102 Tests	80%	98%
Parasitology	66600	ELI.H.A Schistosoma	Bilharziosis	120 Tests	85.7%	98.2%
	66602	ELI.H.A Amoeba	Amoebiasis*	120 Tests	93%	97%
	66604	ELI.H.A Echinococcus	Hydatidosis	102 Tests	93%	94.9%
	66606	ELI.H.A Distoma	Distomiasis	102 Tests	96.1%	96.6%
	66610	ELI.H.A Toxo	Toxoplasmosis	120 Tests	99.2%	97.7%

* Rapid diagnosis of amoebiasis also available in our latex product range: **ELITex Bicolor Amoeba** (Ref. 66608)



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country. Product(s) intended for healthcare professionals only.



: ELITech Microbio (France)



Headquarters
Australia
BENELUX

+33 1 41 45 07 10
+1800 815 098
+31 313 430 500

France
New Zealand
United States

+33 4 83 36 10 82
+0800 555 611
+1 435 0752 6011



ELITechGroup
A BRUKER COMPANY

www.elitechgroup.com
em-marketing@elitechgroup.com