




Microbiology

URIFAST® 2

Antimicrobial
Susceptibility Testing
of Urinary *Enterobacteriaceae*

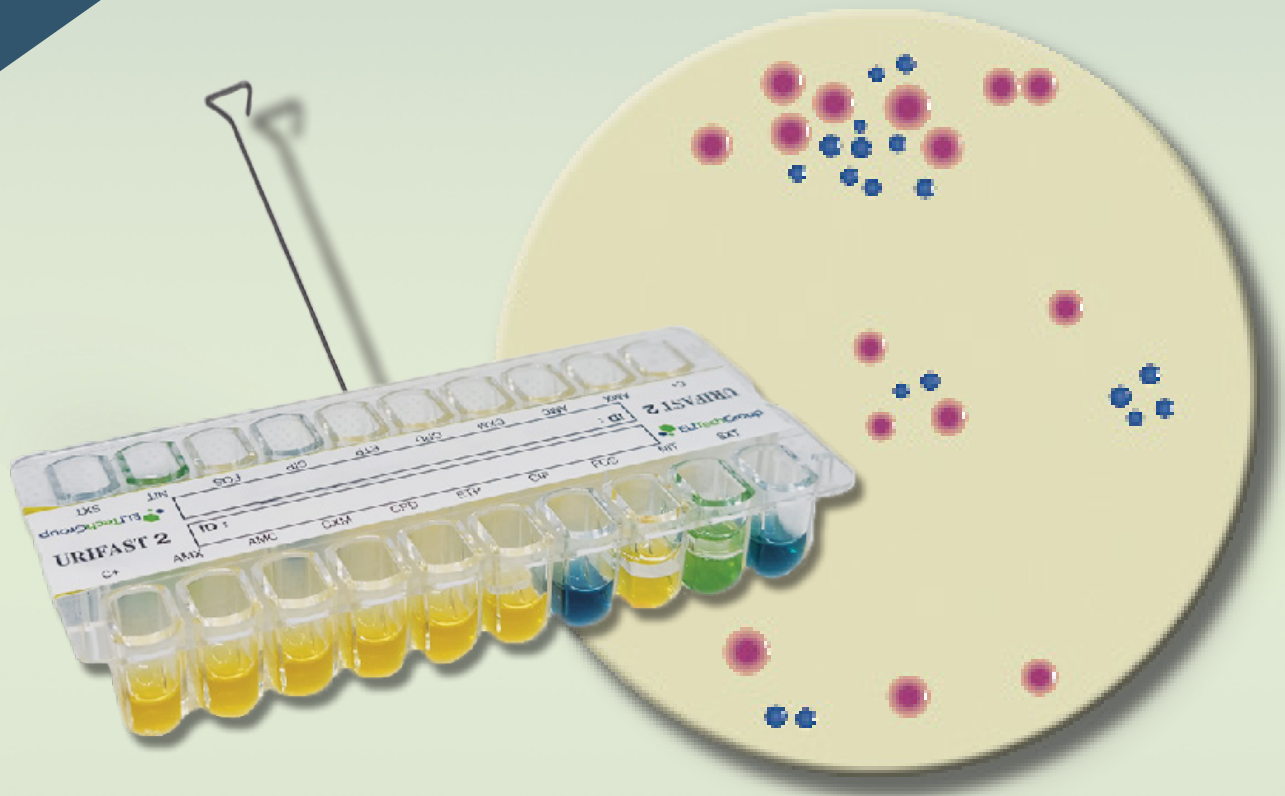

MANUAL, ROUTINE, RAPID TEST

- ◆ Results within 4.5 hours
- ◆ Easy-to-use and cost-effective
- ◆ Visual, colorimetric reading
- ◆ "Presto ABG®" standardization
- ◆ Antibiotic concentrations compliant with EUCAST* recommendations

► BENEFITS

◆ Fast

- ◆ Handling time: 1 minute
- ◆ First results within 4.5 hours

◆ Ease-of-use

- ◆ Standardization of the inoculum with the PRESTO ABG[®]:
 - No suspension medium
 - Direct inoculation of the M4H+
 - One isolated colony required
- ◆ Reading by indicator color change

◆ Cost-effective

- ◆ No instrument required
 - No densitometer
 - No reading apparatus
- ◆ Kit with trays, M4H+ medium and R4H+ reagent
- ◆ Divisible tray
- ◆ Long shelf-life

◆ Reliable

- ◆ Proven standardization
- ◆ Clear visual reading

◆ Performance

- ◆ Antibiotics (AMX/AMC/CXM/CPD/ETP/CIP/FOS/NIT/SXT) compliant with EUCAST recommendations

◆ Reagents


- ◆ **URIFAST[®] 2** trays: divisible tray (2 x 10 wells) for 2 tests
Dehydrated antibiotics at their lower critical concentration
- ◆ **M4H+** medium: vial of modified Mueller-Hinton medium containing bromothymol blue
- ◆ **R4H+** reagent: additive to be used when testing *Proteus mirabilis* strains

► METHODOLOGY

1 Inoculation

Prick the PRESTO ABG[®] right through the colony and the agar

Prick vertically

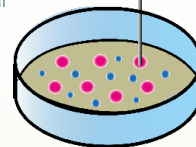


Inoculate the M4H+ medium



2 Distribution into the wells

100 µL per well

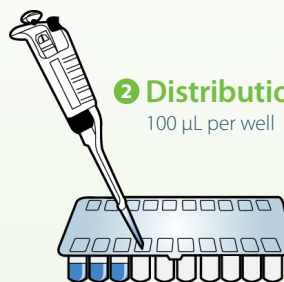

3 Incubation

4.5 hours at 35-37°C

4 Reading

Yellow or green: **growth**
Blue: **absence of growth**



► PRODUCT RANGE

REF.	NAME	QTY	DESCRIPTION
22297	URIFAST [®] 2	20 tests	Trays, M4H+ inoculation medium, R4H+ reagent



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.

Headquarters

Australia
BELUX
Brazil
France
Italy

+33 1 41 45 07 10

1800 815 098
+32 92 820 531
+55 27 3025 1415
+33 4 83 36 10 82
+39 011 67 61 913

New Zealand
Serbia
The Netherlands
UK
United States

0800 555 611
+381 11 2467119
+31 313 430 500
+44 1442 869320
+1 435 0752 6011



ELITech Microbio (France)



www.elitechgroup.com
info@elitechgroup.com