1. INOCULATION DU MILIEU M4H+

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1. INOCULATION OF THE M4H+ MEDIUM

**Do not flame
the PRESTO ABG**

Only use trays and reagents when brought up to room temperature

**Prick up
vertically**

Pick an isolated colony (a minimum of 1mm), prick the PRESTO ABG right through the agar until it touches the base of the dish.

Pick up colony from TSA, sheep blood, chromogenic agar, CLED or Mac Conkey plate

- Place the inoculum into the M4H medium and mix by swirling the PRESTO ABG during 5 sec minimum

- Close the flask of M4H medium and shake vigorously turning the vial successively upside down



2. INOCULATION OF THE TRAY

- Remove the adhesive
- Distribute 100 μ L of inoculated M4H+ medium per well.


If *Proteus mirabilis* is present, before incubation add : 25 μ L of R4H+ to wells AMX, AMC, CXM, CPD and ETP

Add **two** drops of paraffine oil in NIT well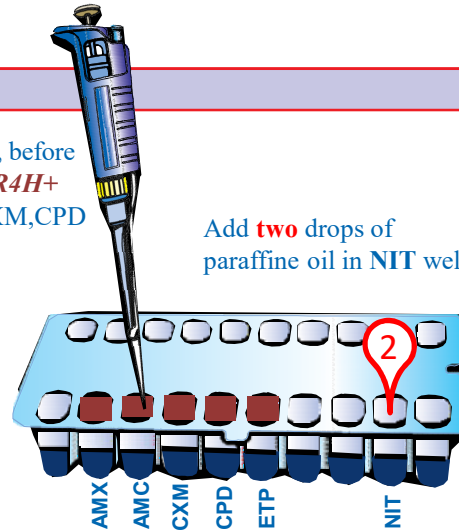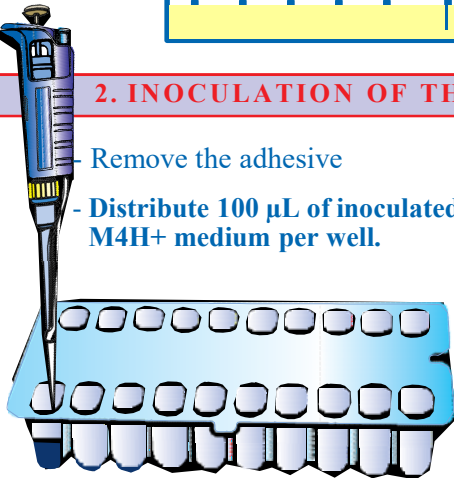

3. INCUBATION OF THE TRAY


Incubate with closing system 4h30 to 6h30 at 34°C to 37°C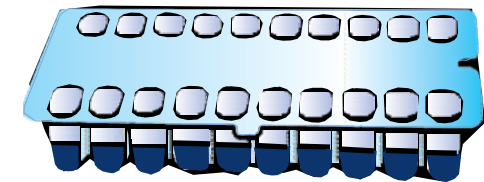

4. READING AND INTERPRETATION

Validation : Positive control = yellow or yellow/green, dark green

Interpretation (wells containing antibiotic)
- Yellow, green or dark green : **Bacterial growth**
- Blue, dark blue : **Absence of bacterial growth**

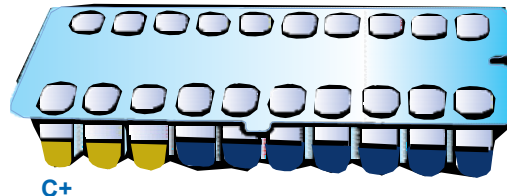C+

Interpret the results referring to the color chart CPD 0138

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Make sure that the result of the antibiotic susceptibility test respects the strain's natural resistance.

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