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2 - Clinical and Laboratory Standards Institute (CLSI). 2011 Methods for Antimicrobial Susceptibility Testing for Human Mycoplasmas; Approved Guideline. CLSI Document M43-A. Vol.31 - N°19.

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6 - WAITES KEN B, DONNAM. CRABB, and LYNN B. DUFFY. 2008. Comparative In Vitro Activities of the Investigational Fluoroquinolone DC-159a and Other Antimicrobial Agents against Human Mycoplasmas and Ureaplasmas. ANTIMICROBIAL AGENTS AND CHEMOTHERAPY, Vol. 52, No. 10, 3776-3778.

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Urogenital mycoplasma diagnosis
MYCOFAST® Screening Revolution
 Screening and differentiation
 50 tests (REF 00063)

COMPLEMENT MYCOFAST® Revolution 2
 Enumeration, identification and susceptibility testing
 25 tests (REF 00082)

UMMt Revolution
 50 tests (REF 00061)

CPB 0396-2_EN-2018-03

For *in vitro* diagnostic use only, for professional use only

1 - INTENDED USE

MYCOFAST Screening *Revolution* (REF 00063) has been designed for the screening and differentiation of *Ureaplasma urealyticum* / *Ureaplasma parvum* (Uu) and *Mycoplasma hominis* (Mh) in various clinical specimens. This kit should be used in association with the media contained in the UMMt *Revolution* kit (REF 00061).

In the case of positive screening, analysis can be completed with trays contained in the COMPLEMENT MYCOFAST *Revolution 2* kit (REF 00082) allowing the enumeration and identification of Uu and/or Mh as well as the antimicrobial susceptibility testing according to the recommendations of the CLSI (Clinical and Laboratory Standards Institute) (2).

2 - INTRODUCTION

Mycoplasmas that include several species that have been identified in humans, all belong to the mollicutes class. They differ from other bacteria in their lack of a cell wall and hence a natural resistance to β -lactams, as well as by the presence of a membrane rich in sterol obtained through their adhesion to eukaryotic cells. Since mycoplasmas are relatively fragile, they will only grow in acellular culture in the presence of various growth factors and at an optimal temperature of 37°C (4). Most human mycoplasmas are commensal. *U. urealyticum* and *M. hominis* are the most commonly encountered species that have been isolated from the urogenital tract. *U. urealyticum* species are divided into two biovars: *U. urealyticum* and *U. parvum* (Uu).

Uu and Mh can be pathogenic. They are responsible for male genital infections (non-gonococcal urethritis, epididymitis, prostatitis, infertility); female genital infections (bacterial vaginosis, endometritis, salpingitis); fertility problems (chorioamnionitis, post-partum endometritis, preterm birth, spontaneous abortion), neonatal problems (low birth weight, respiratory and neurological infections, bacteremias, abscesses); extragenital infections (septic arthritis, reactive arthritis, other infection loci) (1).

The diagnosis of mycoplasma infections depends upon the determination of the pathological threshold, followed by enumeration. The resistance of Uu/Mh to certain drugs necessitates antimicrobial susceptibility testing (5, 6). The drugs tested and the interpretation criteria are adapted for the treatment of infections caused by mycoplasmas encountered in the urogenital tract or in extragenital sites (2).

3 - PRINCIPE

MYCOFAST Screening *Revolution* is a liquid method based on the ability of Uu and Mh to metabolize urea and arginine respectively. The mycoplasma growth results in a colour change of the medium, containing phenol red indicator, from yellow-orange to red. This colour change is due to liberation of ammonia resulting in an alkaline pH of the medium.

Mycoplasma growth thus viewed enables:

- detection and differentiation; then, if a positive result is obtained:
- the enumeration of mycoplasma based on the rate of urea or arginine hydrolysis, which is proportional to the number of germs contained in the sample.
- the identification based on the sensitivity or otherwise of the germ to three antimicrobial agents.
- the Uu and Mh susceptibility testing to antimicrobial agents.

4 - REAGENTS

Description	Amount		
	ref 00061	ref 00082	ref 00063
UMMt : Vial of 3 mL mycoplasma broth with antimicrobial agents and preservative solution. pH : 6.0 ± 0.1	50		
MYCOFAST SCREENING Revolution : Divisible tray of 10 wells for 5 tests, individually packed in an aluminium sachet with an integrated desiccant			10
Etiquettes : Sheet of 5 divisible labels			10
S.Mh. : Mycoplasma hominis growth activator (4.5 mL)			1
MYCOFAST Revolution 2 : Tray of 24 wells for 1 test, packed in an aluminium sachet with an integrated desiccant		25	
Closing System : Protective translucent plastic lid for MYCOFAST <i>Revolution</i> tray		25	

MYCOFAST Screening *Revolution* tray

Tray consisting of 5 rows of 2 wells: a *Ureaplasma urealyticum* (Uu) well containing lincomycin and urea and a *Mycoplasma hominis* (Mh) well containing erythromycin and arginine.

MYCOFAST *Revolution 2* tray

The MYCOFAST *Revolution* tray contains in each of the 20 wells the dehydrated culture medium (foal serum, yeast extract, cysteine, arginine, urea, phenol red, antibiotics, pH: 6.1 ± 0.1) and comprises 4 parts: The MYCOFAST *Revolution 2* tray, in each of the 24 wells, contains dehydrated mycoplasma culture medium (foal serum, yeast extract, cysteine, arginine, urea, phenol red, antibiotics, pH: 6.1 ± 0.1) and includes 2 separate parts:

- the part intended for enumeration and susceptibility testing of the Uu species (wells identified on the label as black).
- the part intended for enumeration and susceptibility testing of the Mh species (wells identified on the label as red).

Diagnosis of Uu species (black part of tray):

- Wells 1/2/3** : Identification and enumeration of Uu at 10³, 10⁴ and $\geq 10^5$ CCU/mL (buffered solution and lincomycin inhibiting Mh growth).
Wells 4/5 : Evaluation of Uu susceptibility to Levofloxacin (LVX) at 2 / 4 μ g/mL
Wells 6/7 : Evaluation of Uu susceptibility to Moxifloxacin (MXF) at 2 / 4 μ g/mL
Wells 8/9 : Evaluation of Uu susceptibility to Erythromycin (ERY) at 8 / 16 μ g/mL
Wells 10/11 : Evaluation of Uu susceptibility to Tetracycline (TET) at 1 / 2 μ g/mL
Wells 12/13 : Evaluation of Uu susceptibility to Doxycycline (DOX) at 1 / 2 μ g/mL

Diagnosis of Mh species (red part of tray):

- Wells 14** : Identification and enumeration of Mh at $\geq 10^4$ CCU/mL
Wells 15/16 : Evaluation of Mh susceptibility to Doxycycline (DOX) at 4 / 8 μ g/mL
Wells 17/18 : Evaluation of Mh susceptibility to Levofloxacin (LVX) at 1 / 2 μ g/mL
Wells 19/20 : Evaluation of Mh susceptibility to Moxifloxacin (MXF) at 0.25 / 0.5 μ g/mL
Wells 21/22 : Evaluation of Mh susceptibility to Clindamycin (CLI) at 0.25 / 0.5 μ g/mL
Wells 23/24 : Evaluation of Mh susceptibility to Tetracycline (TET) at 4 / 8 μ g/mL

	15	16	17	18	19	20	21	22	23	24		
	DOX		LVX		MXF		CLI		TET			
	4	8	1	2	0.25	0.5	0.25	0.5	4	8		
	MYCOFAST® Revolution 2											
14	Mh									2	DOX	13
1	Uu									1		12
	10 ³											
	Uu	2	4	2	4	8	16	1	2			
	10 ⁴	LVX		MXF		ERY		TET				
	$\geq 10^5$	2	3	4	5	6	7	8	9	10	11	

5 - PRECAUTIONS

The reagents are intended solely for *in vitro* use and must be handled by authorized personnel.

The patient samples and inoculated reagents are potentially infectious; they must be handled with caution, in observance of hygiene rules and the current regulations for this type of product in the country of use. Reagents containing raw materials of animal origin must be handled with caution.

Do not use reagents after the expiry date.

Do not use reagents that have been damaged or that have been poorly conserved before use.

A positive result with the MYCOFAST method indicates colonization by urogenital mycoplasmas, but cannot alone be used to make a clinical diagnosis. This must be made by a doctor according to biological results and clinical signs.

6 - SAMPLE COLLECTION AND HANDLING

6.1 Sample collection

Cervicovaginal sample collection

Use only a Dacron or rayon swab or a cytobrush to collect samples.

The cervix should be carefully cleaned with a swab, to remove secretions, before collecting the sample with a new swab. As mycoplasmas adhere strongly to mucous cells, the mucous lining should be vigorously scrubbed to obtain a rich specimen.

Urethral sample collection :

Clean the meatus and swab or scrape the area to obtain cells.

Sperm, Urine

Collect sperm or first micturition in a sterile tube or bottle.

6.2 Transport in UMMt medium

Swab samples: Place the swab in a vial of UMMt medium.

Liquid samples: Inoculate a vial of UMMt medium with 300 μ L of homogenized fluid.

6.3 Conservation in UMMt medium

The inoculated UMMt medium may be kept for 20 hours at room temperature (18-25°C) or 56 hours at 2-8°C. For storage during 3 days at -20°C, first add 2 drops of "MYCOPLASMA Stabilizer".

7 - PREPARATION AND STORAGE OF REAGENTS

All the reagents are ready-to-use. The vials may be stored at 2-8°C, in their original packaging until the expiry date shown on the kit.

Should only one or two, three, or four rows of (Uu) (Mh) wells be used, the remaining MYCOFAST Screening *Revolution* tray may be stored for 4 weeks at 2-8 °C in its original packaging and hermetically resealed.

The UMMt medium may be stored temporarily (3 months) at room temperature but is more stable at 2-8°C.

The S.Mh supplement is stable for 3 months after opening. Do not freeze the reagents contained in the kit.

8 - MATERIAL REQUIRED BUT NOT PROVIDED

Sample collection (swabs, cytobrushes, sterile containers for liquid samples)
 MYCOPLASMA Stabilizer (REF 00064)
 Pipettes and tips; Incubator at 37°C ± 1°C
 Waste container for contaminated waste and mineral oil

9 - METHOD

Allow the reagents to reach room temperature (20-30 minutes).

9.1 SCREENING - MYCOFAST Screening *Revolution* tray

- Prepare as many rows of wells as samples to be tested.
 - If required separate one or several rows of (Uu)/(Mh) wells with the aid of the marks found on the tray.

9.1.1 Seeding of the UMMt *Revolution* medium

Seed the UMMt medium with a swab or 300 µL of liquid sample (§6.2). Mix well.

9.1.2 Inoculation of Uu/Mh wells

- Distribute successively:
 (Uu) well: 100 µL of seeded UMMt medium.
 (Mh) well: 100 µL of seeded UMMt medium.
 50 µL of Mh supplement.
 - Add 2 drops of mineral oil to the two wells.
 - Cover the wells with the divisible labels and label the sample in order to identify it.
 - **Store excess seeded UMMt medium at 2-8°C** in order to continue the analysis in case of positive screening.

9.1.3 Incubation of Uu/Mh wells

Incubate the tray at 37°C ± 1°C for 24 hours. Tray incubation can be extended for up to 48 hours only in the case of liquid samples that are negative after 24 hours.

9.1.4 Reading and interpretation of Uu/Mh wells

- Check that the 2 (Uu) (Mh) wells are limpid. A cloudy appearance in a well indicates bacterial contamination. In this case repeat the analysis. - Observe the colour change of the medium in the Uu and Mh wells:
 Uu wells are orangey or red: presence of *Ureaplasma urealyticum*
 Mh wells are orangey or red: presence of *Mycoplasma hominis* Uu / Mh wells are yellow: absence of mycoplasma
In the case of positive screening continue diagnosis with the MYCOFAST *Revolution* 2 tray.

9.2 ENUMERATION, IDENTIFICATION AND SUSCEPTIBILITY TESTING

9.2.1 Inoculation of the MYCOFAST *Revolution* 2 tray

Remove the adhesive film by pulling on the tab and add the following to the wells of each row:
 Wells 1-24 100 µL of inoculated UMMt medium
 Wells 1-24 2 drops of mineral oil
 Cover the seeded tray with the "closing system".
 Label the sample.
Store excess UMMt medium at 2-8°C for at least 48 hours for possible verification.

9.2.2 Incubation of the tray

Incubate the tray at 37°C ± 1°C for 24 hours.
 For Uu and Mh enumeration, read the results in 24 hours. Tray incubation can be extended for up to 48 hours only in the case of liquid samples that are negative after 24 hours.

9.2.3 Reading and interpretation

Check that all the wells in the row are limpid. A cloudy appearance in a well indicates bacterial contamination. In this case repeat the analysis. The results are read by the colour obtained in the different wells. Urogenital mycoplasma growth is indicated when the medium turns red (alkaline). The medium remains yellow when no growth of urogenital mycoplasma occurs. An orangey coloration should be considered as a positive test (rate limit).

For the interpretation of the results refer to the results sheet.

Enumeration (wells 1, 2, 3 and 14)

Mark the wells that have turned red and interpret:
 1 Uu value 10³ CCU/mL
 1 and 2 Uu value 10⁴ CCU/mL
 1, 2 and 3 Uu value ≥10⁵ CCU/mL
 14 Mh value ≥ 10⁴ CCU/mL

The pathological thresholds usually quoted for *U. urealyticum* are: ≥10⁴ CCU/mL for a urethral specimen or endotracheal specimen, ≥10³ CCU/mL in a first urine stream or sperm (although a new local recommendation mentions a threshold ≥10⁴ CCU/ml for semen (7)). The presence of *M. hominis* at a threshold ≥ 10⁴CCU/mL in an cervicovaginal specimen is abnormal (1, 3).

Susceptibility testing (wells 4 to 13 and 15 to 24)

The red colour change of the medium in the wells containing an antibiotic indicates the presence of bacterial growth and hence resistance to the antibiotic concentration being tested. The yellow colour of the medium indicates the absence of bacterial growth and hence susceptibility to the antibiotic concentration being tested. The strains are characterized as being susceptible or resistant to the antibiotics according to the following criteria defined by the CLSI (2):

Table of MIC (µg/mL) interpretative criteria

Antibiotic		Uu		Mh		Comments
Class	Drug	S	R	S	R	/
Quinolones	Levofloxacin	≤2	≥4	≤1	≥2	/
	Moxifloxacin	≤2	≥4	≤0.25	≥0.5	/
Lincosamides	Clindamycin	/	/	≤0.25	≥0.5	/
Tetracyclines	Tetracycline	≤1	≥2	≤4	≥8	/
	Doxycycline	≤1	≥2	≤4	≥8	/
Macrolides	Erythromycin	≤8	≥16	/	/	Organisms susceptibles to erythromycin will also be susceptible to azythromycin

Help with interpretation:

Susceptibility testing of Uu

Antibiotic	LVX			MFX			ERY			TET			DOX		
	2	4	int*	2	4	int*	8	16	int*	1	2	int*	1	2	int*
Profile	-	-	S	-	-	S	-	-	S	-	-	S	-	-	S
	+	-	R	+	-	R	+	-	R	+	-	R	+	-	R
	+	+	R	+	+	R	+	+	R	+	+	R	+	+	R

Susceptibility testing of Mh

Antibiotic	DOX			TET			ERY			MFX			LVX		
	1	2	int*	1	2	int*	8	16	int*	2	4	int*	2	4	int*
Concentration (µg/mL)	1	2	int*	1	2	int*	8	16	int*	2	4	int*	2	4	int*

Profile	-	-	S	-	-	S	-	-	S	-	-	S	-	-	S
	+	-	R	+	-	R	+	-	R	+	-	R	+	-	R
	+	+	R	+	+	R	+	+	R	+	+	R	+	+	R

int*: interpretation

The strain is said to be susceptible when its growth is inhibited by the higher and lower critical concentrations of the antibiotic.

The strain is said to be resistant when its growth is inhibited by the higher critical concentration of the antibiotic, but not the lower critical concentration or when its growth is not inhibited by either the higher or the lower critical concentrations of the antibiotic.

For moxifloxacin only one concentration is tested for Uu and Mh. *M. hominis* strains are innately resistant to macrolides (14 -15 carbon atoms), including erythromycin.

In some patient populations, tetracycline resistance is as high as 45% for Uu and 39.6% for Mh (2). Uu/Mh quinolone (5, 6) and clindamycin resistance have been described but the prevalence is not known.

10 - PARTICULAR CASES

For high Uu and Mh levels, the content of all the wells on the tray has turned red. It is recommended that the sample be diluted in order to obtain more specific results. In this case, proceed as follows:

Inoculate a new UMMt 3mL vial with 300 µL of the original UMMt medium stored at 2-8°C (see § 9.1).

Inoculate a new tray with the new inoculated UMMt medium.

Take the dilution (1:10) into account in the interpretation of the enumeration results.

If necessary, confirm the presence of mycoplasmas on an A7 agar plate by re-isolating from the original UMMt medium stored at 2-8°C (§ 9.1). A non-constant incubation temperature or <36°C (frequent opening and poor temperature heterogeneity of the incubator) can slow down the mycoplasma growth kinetics.

11 - QUALITY CONTROL

Quality control can be carried out from the lyophilized *U. urealyticum* strain of the MYCOPLASMA CONTROL kit (REF 00900)) or from a lyophilized reference strain (*U. urealyticum* ATCC 27815 or *M. hominis* ATCC 23114) previously calibrated at 104-5 CCU/mL.

Screening :

Inoculate the two wells of the MYCOFAST Screening *Revolution* tray and perform the test as indicated in these instructions (§ 9.1).

Expected results: for strain Uu : Uu (+) et Mh (-) ; for strain Mh: Uu (-) et Mh (+).

Enumeration, identification and susceptibility testing:

Inoculate the MYCOFAST *Revolution* 2 tray and perform the test as indicated in these instructions (§ 9.2).

Expected results (ATCC):

	MYCOFAST <i>Revolution</i> 2									
	Uu 10 ³	Uu 10 ⁴	Uu ≥10 ⁵	Mh ≥10 ⁴	LVX	MXF	ERY	TET	DOX	CLI
Strain Uu ATCC 27815	+	+	+/-	NA	S	S	S	S/R	S	NI*
Strain Mh ATCC 23114.	-	-	-	+	S/R	S	NI*	S	S	S

NI* (Non Interpretable) : natural resistance

12 - LIMITATIONS OF THE PROCEDURE

12.1 - Screening:

The MYCOFAST Screening *RevolutioN* tray allows detection at a threshold of $\leq 10^3$ UCC/mL and does not enable numeration. The numeration obtained with the MYCOFAST *Revolution* tray can appear negative following positive screening.

12.2 - Identification, enumeration and susceptibility testing

Some bacteria that are present in quantities of $>10^{6-7}$ CFU/mL and contain urease may cause all the wells in the tray to change colour. The presence of these can be verified by re-isolating on chocolate agar from the original UMMt medium stored at 2-8°C (see § 9.1).

An alkaline sample pH (pH > 8) may lead the UMM medium to change colour. Should this occur, dilute the sample (1:10) in fresh UMM medium and interpret the results taking the dilution into account.

A sample with an acidic pH (pH ≤ 5) can slow down the appearance of the colour change.

A sample containing blood may cause a colour change in the wells of the MYCOFAST *RevolutioN* 2 tray and could be interpreted as a positive result. In this case dilute the sample (1:10) in another UMMt medium and interpret the results, taking into account the dilution.

A sample with a low mycoplasma load ($<10^3$ CCU/mL) may lead to a random colour change in the different wells of the tray.

As for all germ detection methods, the quality of the sample can influence the test result. A negative test does not therefore necessarily indicate the absence of infection.

13 - PERFORMANCES

13.1 Screening and differentiation

MYCOFAST Screening *RevolutioN* tray

A comparative study was carried out using dry swabs on clinical vaginal samples (n= 40 for both Uu and Mh species).

The results obtained with MYCOFAST Screening *RevolutioN* were compared with those obtained with the liquid micro-dilution method.

- For the screening of both Uu and Mh species, there was 97.5% concordance.

For Uu. at 10^3 CCU/mL, 1 sample that was positive with the liquid micro-dilution method appeared negative with the MYCOFAST Screening *RevolutioN*. However, it is important to underline that the 10^3 CCU/mL concentration corresponds to an infra-pathological threshold usually quoted for Uu. For Uu and Mh, the global agreement for the supra-pathological threshold is 100%.

- Differentiation of the mycoplasma species from all tested samples were correctly identified in the corresponding Uu and Mh wells of the MYCOFAST Screening *RevolutioN* tray.

13.2 Identification and enumeration

Direct method MYCOFAST *RevolutioN* 2 tray

% of overall agreement	Uu	Mh	Uu/Mh
isolated strains (threshold $\leq 10^3$ CCU/mL) (see § 14.1.1)	97.4	NA*	NA*
isolated strains (threshold $\geq 10^4$ CCU/mL) (see § 14.1.1)	93.4	93.4	93.4
vaginal samples (see § 14.1.2)	100	100	100
urinary clinical specimens (see § 14.1.2)	93.2	96.6	94.9

NA* (Not Applicable)

13.2.1 Isolated strains

A comparative study was carried out with 21 isolated strains (ATCC strains and collection strains) tested separately (Uu or Mh) with several concentrations (76 tests in total).

The results obtained were compared with those obtained with the micro-dilution enumeration method.

For interpretation with a pathological threshold set at 10^3 CCU/mL, the overall agreement for Uu is 97.4% (we listed 2 false positives at 10^2 CCU/mL with the micro-dilution enumeration method).

For interpretation with a pathological threshold of 10^4 CCU/mL; the overall agreement for Uu is 93.4% (we recorded 5 false positives at 10^3 CCU/mL with the micro dilution enumeration method). The overall agreement for Mh is 93.4% (we listed 5 false positives, 4 at 10^3 CCU/mL and one at 10^2 CCU/mL with the micro dilution enumeration method).

The overall Uu + Mh agreement is 93.4%.

13.2.2 Clinical samples

An initial comparative study was performed using vaginal clinical specimens (n =23) on dry swabs. The results obtained with MYCOFAST *RevolutioN* 2 were compared with the liquid micro-dilution enumeration method. The overall agreement for Uu and Mh is 100%.

A second comparative study was performed on urinary clinical specimens (n=88).

The results obtained with MYCOFAST *RevolutioN* 2 were compared with those obtained with with the liquid micro-dilution enumeration method.

The overall agreement for Uu is 93.2% (we listed 1 false negative at 10^4 CCU/mL with with the liquid micro-dilution enumeration method and 5 false positives at 10^2 CCU/mL with with the liquid micro-dilution enumeration method).

The overall agreement for Mh is 96.6% (we identified 3 false positives at $10^2 - 10^3$ CCU/mL with the liquid micro-dilution enumeration method).

The overall agreement for Uu and Mh is 94.9%.

13.3 Susceptibility testing

A comparative study was carried out in a national reference laboratory between the method for determining the minimum inhibitory concentrations (MIC) in liquid medium and the MYCOFAST *RevolutioN* 2.

The tested strains (7 *U. urealyticum*, 11 *U. parvum* and 16 *M. hominis*) were reference strains, wild-type clinical strains or strains with acquired resistance. Each strain was tested at 10^3 , 10^4 and 10^5 CCU/mL dilutions in UMMt 3 mL.

For 10^4 and 10^5 CCU/mL rates, results were read and interpreted after 24 hours of incubation.

For 10^3 CCU/mL rate, results were read and interpreted after 48 hours incubation in case of negative test in 24 hours.

The results of both methods were interpreted as susceptible (S) or resistant (R) according to CLSI recommendations.

The overall agreement for *Ureaplasma urealyticum* / *Ureaplasma parvum* is: 95.5%.

The overall agreement for *Mycoplasma hominis* for rates at 10^4-10^5 CCU/mL is: 100%.

Agreement	<i>Ureaplasma urealyticum</i> / <i>parvum</i> (n=40)					<i>Mycoplasma hominis</i> (n=28)				
	TET	DOX	MOX	LVX	ERY	TET	DOX	MOX	LVX	CLI
	34	38	40	39	40	28	28	28	28	28
ME	5 ^a	0	0	0	0	0	0	0	0	0
VME	1 ^b	2 ^c	0	1 ^d	0	0	0	0	0	0

ME: Major Error, VME : Very Major Error

- ^a : 1 discrepancy at 10^3 CCU/mL (MIC of reference 0.5 μ g/mL), 4 discrepancies at 10^5 CCU/mL (MIC of reference 0.5 - 1 and 8 μ g/mL).
^b : 1 discrepancy at 10^5 CCU/mL (MIC of reference 8 μ g/mL).
^c : 1 discrepancy at 10^3 CCU/mL (MIC of reference 8 μ g/mL); .1 discrepancy at 10^5 CCU/mL (MIC of reference 2 μ g/mL)
^d : 1 discrepancy at 10^5 CCU/mL (MIC of reference 4 μ g/mL).

14 - ELIMINATION DES DECHETS

Waste should be disposed of in accordance with the hygiene rules and current regulations for this kind of product in the country of use.

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MYCOFAST® is a trademark of ELITech MICROBIO

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