



Microbiology

ELI ATB BOVINE
ELI ATB PETS



SIMPLE AND QUICK TREATMENT ORIENTATION

Trays for testing bacterial susceptibility to reference antibiotics for cats and dogs or cattle.

- ◆ **Reference antibiotics** at concentrations recommended by the veterinary entity of the **CASFM*** and/or defined by the **CLSI**** and validated in animal epidemiology.
- ◆ **Quick and easy-to-use** in 3 minutes.
- ◆ **Visual interpretation** and results in 6 hours (24h for *Enterococcus spp* and *Pseudomonas spp*).

*Antibiogram Committee of the French Society of Microbiology

** Clinical and Laboratory Standards Institute

ELI ATB PETS & BOVINE

► PRINCIPLE

The ELI ATB method consists of studying the sensitivity of bacteria to antibiotics in liquid medium.

The test is carried out using a bacterial colony isolated on agar (ELI COLOREX VET ID or others), and standardised in a medium containing a coloured indicator.

The bacterial suspension obtained is then uniformly distributed in the wells of an ELI ATB tray containing the antibiotics to be tested.

► COMPOSITION

ELI ATB Pets

11 antibiotics

- Clindamycin
- Marbofloxacin


+ 9
antibiotics
common to
trays

ELI ATB Bovine

12 antibiotics

- Ceftiofur
- Florfenicol
- Penicillin/Novobiocin
- Penicillin
- Amoxicillin/clavulanic acid
- Ampicillin
- Oxacillin
- Erythromycin
- Tetracycline
- Gentamycin
- Enrofloxacin
- Trimethoprim/Sulfamethoxazole


► PROCEDURE

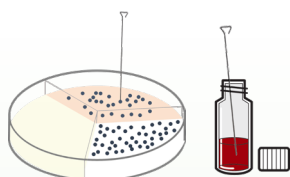

1 Preparation of the inoculum

Prick a colony* with the PRESTO ABG® inoculator and discharge the inoculum into the ELI ES medium for 5 seconds.

**Vertically until it slightly touches the bottom of the box.*

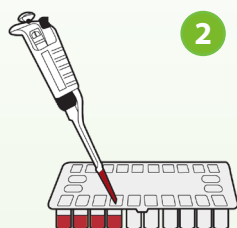

2 Inoculation

Shake the vial and dispense 3 drops of medium per well. Close the tray with a Closing System.

3 Incubation 6h at 37 °C and 24h for *Enterococcus spp* et *Pseudomonas spp*.

4 Read the results

Positive

Bacterial growth
Antibiotic not effective



yellow, orange, fuchsia

Negative

Inhibited growth
Effective antibiotic



red

► PRODUCT RANGE

REF.	NAME	QTY
22804	ELI ATB BOVINE	2 Tests
22805	ELI ATB PETS	2 Tests



WORLDWIDE OFFICES

Please contact your sales representative for terms, conditions and product availability in your country.



: ELITech Microbio (France)

Headquarters
Australia
BENELUX

+33 1 41 45 07 10
+1800 815 098
+31 313 430 500

France
New Zealand
United States

+33 4 83 36 10 82
+0800 555 611
+1 435 0752 6011



ELITechGroup
A BRUKER COMPANY

www.elitechgroup.com
em-marketing@elitechgroup.com