




ELITex Bicolor Mono

S E R O L O G Y R A P I D T E S T

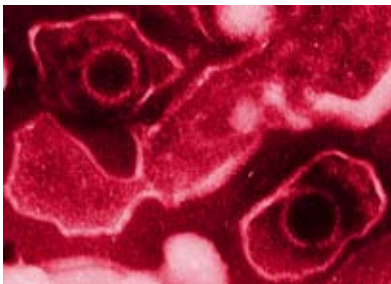

Epstein Barr Virus

Slide agglutination test for rapid detection of **INFECTIOUS MONONUCLEOSIS** specific heterophile antibodies in serum.

- ◆ **Easy reading: bi-colored latex reaction**
- ◆ **Complete kit**
- ◆ **Simple and fast protocol (2 minutes)**
- ◆ **Cost effective test**
- ◆ **Long shelf life**

PRINCIPLE:


Latex particles are coated with specific antigens and added to the test serum. If the specific antibodies are present in the serum, they will agglutinate and be revealed by a **bi-colored** reaction.

Due to the use of specially-treated latex particles, no adsorption of the other heterophile antibodies is necessary.


METHODOLOGY:

S I M P L E A N D F A S T

1. Add 50 µL Serum or Control(s)
2. Add 1 drop Test Latex

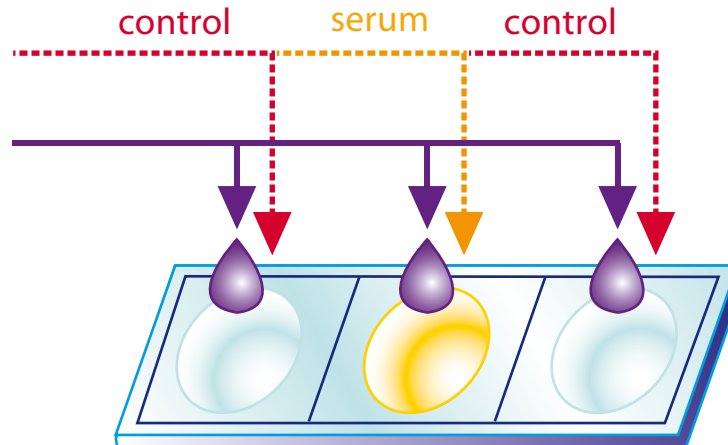

3. Slow rotary motion

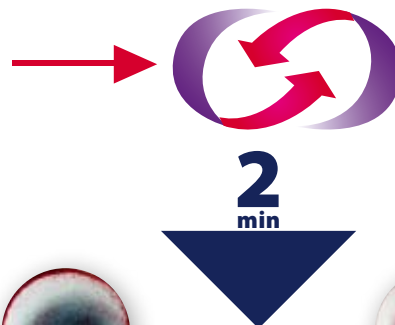

POSITIVE REACTION:

Red agglutinates on a blue background.

INTERPRETATION: Presence of infectious mononucleosis heterophile antibodies.



NEGATIVE REACTION:

Homogeneous and purple suspension.

INTERPRETATION: Absence of infectious mononucleosis heterophile antibodies.



PRODUCT RANGE:

REFERENCE	NAME	QTY	SENSITIVITY	SPECIFICITY
04155	ELITex Bicolor Mono	50 tests	98.4%	99.7%

Distributor Contact Information:

Product not available in the US
Please contact your sales representative for product availability in your country

WORLDWIDE OFFICES:

Belgium T: +32 9 2820 531
Brazil T: +55 27 3025 1415
France T: +33 4 90 17 54 50
Italy T: +39 02 48 40 35 42

Middle East & Africa T: +971 4 375 2744
Serbia T: +381 11 2467119
Switzerland T: +41 26 663 86 60
The Netherlands T: +31 485 511 675

UK T: +44 1268 493377
United States T: +1 609 216 7360
855-ELITECH (855-354-8324)

Headquarters T: +33 1 41 45 07 13
www.elitechgroup.com
info@elitechgroup.com