
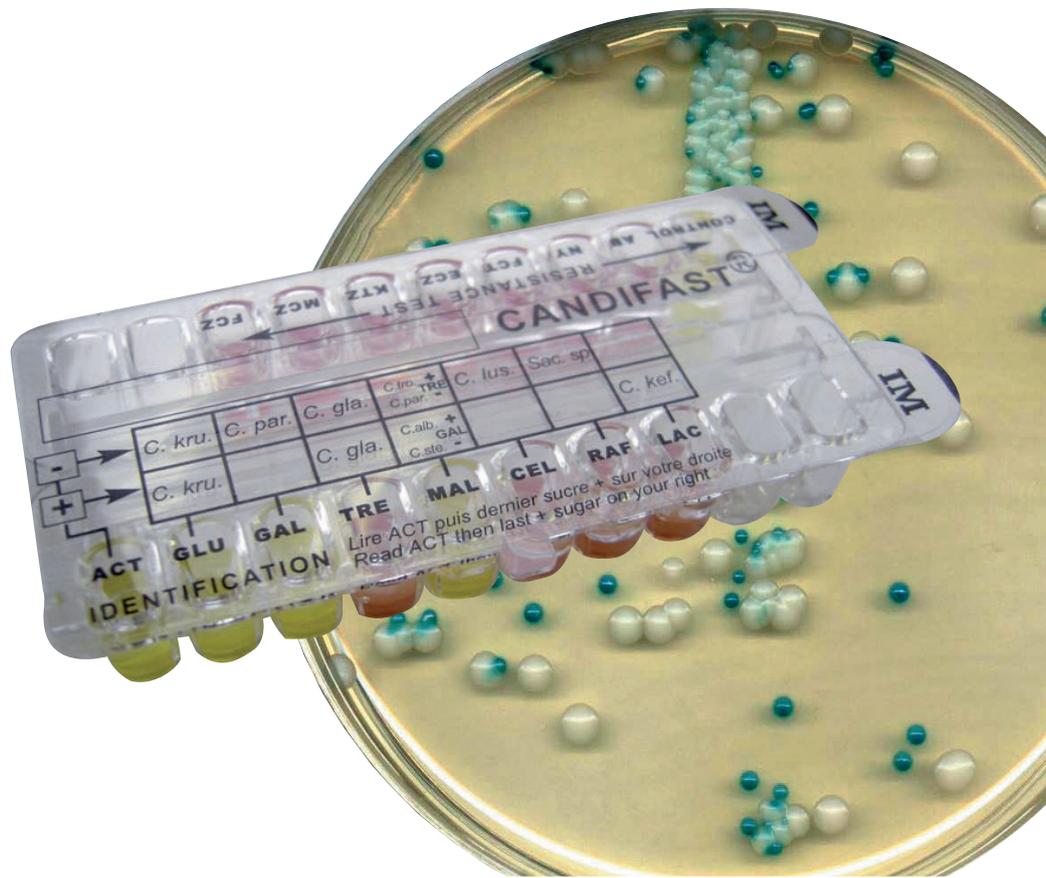


CANDIFAST®

DIAGNOSIS OF THE PATHOGENIC YEASTS

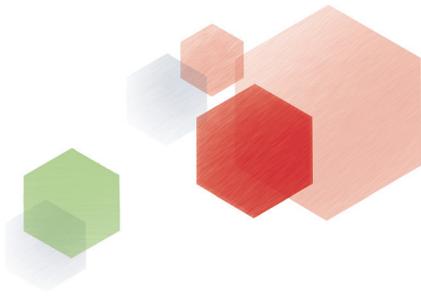

A test combining Identification and Resistance Testing from the same inoculum

- ◆ Convenient
- ◆ Easy to read and interpret
- ◆ Cost effective
- ◆ Suitable for superficial infections

BENEFITS

Convenient

- Same tray combining
 - Identification
 - Antifungal resistance testing
- Same inoculum
- Same incubation at 37°C

Easy to read and interpret

- Spontaneous coloured reactions
- Simplified results interpretation sheet
- Rapid identification directly from the label

Cost effective

- Two-in-one test
- All reagents included in the kit
- Instrument free

Suitable method for superficial infections

Identification

- Identification of the main pathogenic yeasts :
(*C. albicans*, *C. glabrata*, *C. kefyr*, *C. krusei*, *C. parapsilosis*, *C. lusitaniae*, *C. tropicalis*, *Saccharomyces*, *Cryptococcus* or *Trichosporon* or *Rhodotorula*).
- Main identification characters
 - Susceptibility to actidione
 - Fermentation of 7 sugars
 - Hydrolysis of urea

Resistance testing

- Panel of antifungals adapted to superficial fungal infections (Amphotericin B, Nystatine, Flucytosine, Econazole, Ketoconazole, Miconazole, Fluconazole).

Fast results

- Results within 24 hours incubation for most of the yeasts

Products

- **R1:** Buffered agar medium for dilution and Identification
- **R2:** YNB medium for resistance testing
- **TC:** Barium sulphate solution for turbidity control
- **CANDIFAST tray:** 20-well tray, ready to use.

Each tray allows the testing of one sample:


- One identification row of 8 wells
- One antifungal resistance testing row of 8 wells

Catalog


- **CANDIFAST®** (2-in-1 product)
Kit of 30 tests - Ref. 44030
- **CANDIFAST® ES TWIN** (Resistance test only)
Kit of 30 tests - Ref. 44130

METHOD


1 - Preparation of the inoculum in R1 medium

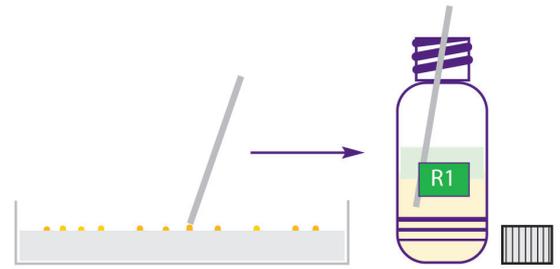

2 - R2 medium inoculation

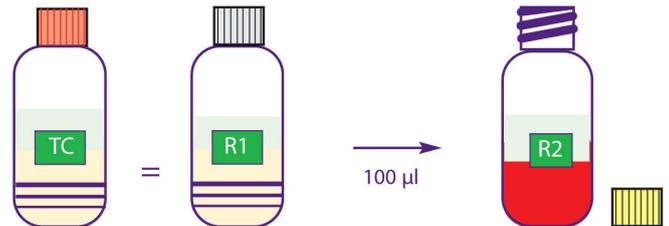

3 - Tray inoculation

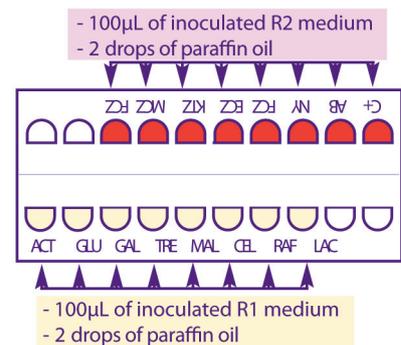

4 - Incubation at 37°C / 24-48h

5 - Read the color change



Colored pictures above are only shown as examples.

DISTRIBUTOR CONTACT INFORMATION

Product not available in the US.
Please contact your sales representative for product availability in your country

 : ELITech Microbio (France)

WORLDWIDE OFFICES

Headquarters
Australia T: +33 1 41 45 07 10
Belgium & Luxembourg T: +32 92 820 531
Brazil T: +55 27 3025 1415
France T: +33 4 83 36 10 82
Italy T: +39 02 48 40 35 42
New Zealand T: +64 0800 555 611

Serbia T: +381 11 2467119
Switzerland T: +41 26 663 86 60
The Netherlands T: +31 313 430 581
UK T: +44 1442 869320
United States T: +1 609 216 7361

 **ELITechGroup**
SOLUTIONS
tailored to your needs
www.elitechgroup.com
info@elitechgroup.com